

REST EASY WITH

Purify **O₃**TM

PORTABLE OZONE CPAP/BiPAP SANITIZER



**DISINFECT WITH
JUST THE TOUCH
OF A BUTTON**



#190-6000

The simple & effective way to clean your CPAP/BiPAP mask and equipment

Powerful, safe hospital grade sanitation cycle utilizes ozone to kill 99.99% of germs and bacteria in just 35 minutes! Lightweight & compact - ideal for use at home and away.

No extra supplies required - sanitizes and stores in the included cleaning/travel case!

Don't take a chance with your health! Sanitize with *Purify* **O₃**

**KILLS 99.99%
OF GERMS
& BACTERIA**



SPECIFICATIONS:

Size/Dimensions: 4.5" x 2.75" x 2"

Weight: 0.5 lb

Cleaning Technology: Ozone (O₃)

Power Source: 4W, Rechargeable Lithium-ion battery

Recommended Frequency of Use:

- Three times per week: mask sanitation
- Once per week: humidification chamber, tubing

Includes: Purify O3 unit, dual purpose sanitizing and travel case, tubing adapter, USB cable and charging adapter, bilingual instruction manual (English and Spanish)

Warranty: 2 years - register at respondo2.com/warranty

FDA Registered

SIMPLE TO USE:

- Empty humidifier chamber and wipe mask/tubing to remove visible residue, attach and or place Purify O3 in sanitizing case or closed container.
- Closed container disinfection process ensures safe and effective equipment sanitation at a higher efficacy than other cleaning methods.
- Push to start - Easy one-touch control. Automatically shuts off after 35 minute cleaning cycle is complete. (Note: Allow ozone to dissipate for additional 90 minutes before use.)
- Recharges quickly via wall outlet or USB port.
- No maintenance, filters, or extra cleaning solutions required.
- May also be used to clean nebulizer mask, tubing, breast pump accessories or other medical devices.

PURIFY O3 CPAP/BiPAP SANITIZER KIT

#190-6000 Purify O3 Sanitizer Kit

REPLACEMENT COMPONENT PARTS

- #190-6002 Dual Purpose Sanitizing/Travel Case
- #190-6003 USB Cable
- #190-6004 Charging Adapter
- #190-6005 Tubing Adapter



#190-6000